

QUICK USER GUIDE


En

!!! Please refer to our website: www.geemarc.com for an up to date user guide, as there may be important updates and changes you need to be aware of !!!

Congratulations on purchasing your Geemarc AMPLIDECT COMBI PHOTO 295 telephone. This telephone offers all the advantages of cordless technology (compliant with Digitally Enhanced Cordless Telecommunications-DECT standard) together with an easy to see big button keypad, ideal for those who with visual impairment. This telephone is hearing aid compatible.

It is important that you read the instructions below in order to use your Geemarc telephone to its full potential.

Wall Mounting

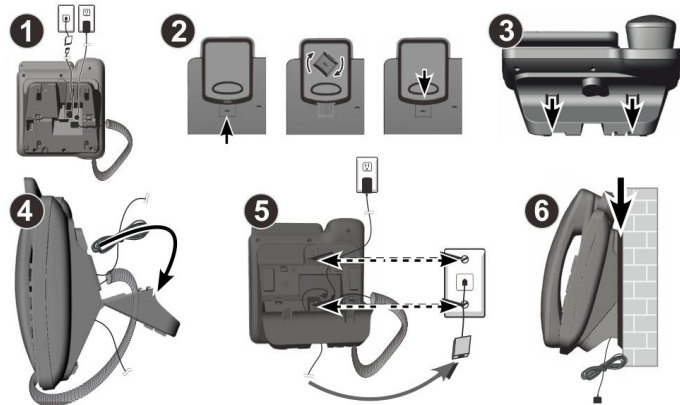

CAUTION: Use only the battery pack provided. There may be a risk of explosion if a wrong type of battery is used for the parent unit. Do not subject the battery pack to extreme high or low temperatures, or low air pressure at high altitude during use, storage or transportation. Disposal of the battery pack into fire or a hot oven, or mechanically crushing or cutting or the battery can result in an explosion. Subjecting the battery pack to an extremely high temperature environment, or extremely low air pressure can result in an explosion, or leakage of flammable liquid or gas.

Battery subjected to extremely low air pressure may result in an explosion or the leakage of flammable liquid or gas.

Normal operating temperature for the product (e.g. 0°C~40°C).

Use only the adapters and battery pack listed below.

Power adapter information: output: 6V DC 0.45A, 2.7W. Vtech Telecommunications Limited.


Model: VT05EEU06045 (EU), VT05EUK06045 (UK)

Rechargeable Battery Pack: Model: NI-MH AAA500x2. 2.4V, 500mAh. Corun.

PACKAGE CONTENTS

1 Telephone base, 1 Corded handset, 1 cordless photo handset, 2 Power adapter, 1 Charger, 1 Telephone line cord, 1 NiMH Battery pack, 1 Quick user guide.

INSTALLATION

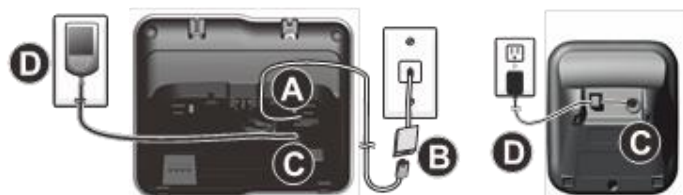

Connecting the telephone

Connect the telephone cable to the telephone line socket (A on the base) and the telephone wall socket (B).

Connect the handset curly cord to the base.

Connect the cable from the power adapter to the small round socket

1

on the telephone base (C), then plug-in the power adapter to a mains outlet socket (D).

Note: Only use the power adapters supplied with the phone.

Connecting the battery

Remove the battery cover from the handset. Connect the short lead from the battery pack to the socket inside the battery compartment.



Carefully fit the battery pack as shown.

To replace the battery cover, carefully align it flat against the battery compartment, then slide it towards the centre of the handset until it clicks into place


Charge the handset battery pack

To charge the battery pack, place the handset on the pod (charger).

Note: Charge the battery pack for at least 16 hours when charging for the first time.

OVERVIEW

Telephone base

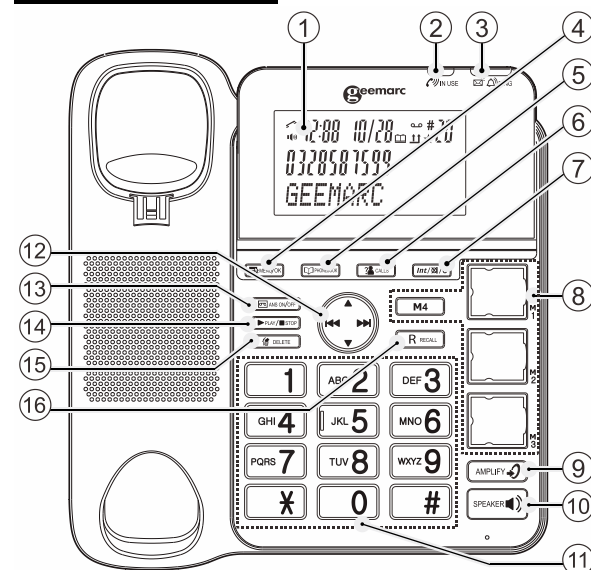

1- LCD DISPLAY

2- IN USE (): On during a call.

3- RING light ()

Flashes when there is an incoming call.

On when the answering machine is switched on. Flashes when there are new memos or messages left on the answering machine.

4- MENU/OK ()

In idle mode: Press to access the main menu.

In sub-menu mode: Press to confirm the selection.

In Redial list / Call list: Press to store the number into Phonebook.

5- PHONEBOOK ()

In Idle mode: Press to access the phonebook.

6- CALLS (CALL LIST) ()

In Idle mode: Press to access the call list.

7- INT/MUTE/CLEAR ()

In main menu mode: Press to go back to idle screen.

In sub-menu mode: Press to go back to previous level.

In sub-menu mode: Press and hold to go back to idle screen.

In editing / predialing mode: Press to clear a character / digit.

In editing / predialing mode: Press and hold to delete all the character / digits

During a call: Press to mute / unmute the microphone.

In ringing: Press to silence the base ringer.

In idle mode: Press to intercom another system handset.

8- MEMORY KEYS (3 PHOTO SPEED DIAL KEYS;

1 MEMORY KEY - M4)

In idle mode: Dial your phone number and Press and hold the memory key until to hear a beep.

To use: Press to dial a number stored in the memory key.

9- VOLUME BOOST ()

During a call: Press to activate / deactivate the VOLUME BOOST function.

10- SPEAKERPHONE ()

In Idle mode or Phonebook / Call list / Redial list view mode: Press to make a call with speakerphone.

11- ALPHANUMERIC KEYPAD, STAR (*), HASH (#)

In predial or editing mode: Press to insert a digit.

0 key in predial or editing mode: Press and hold to insert a pause.

12- NAVIGATION KEYS

UP ()

In menu mode: Press to scroll up the menu items.

In Phonebook list / Redial list / Call list: Press to scroll up the list.

In editing mode: Press to move the cursor one character to the left.

During a call or TAM message playback: Press to increase the volume.

REPEAT/OGM ()

In Idle mode: Press and hold to record OGM.

Press to playback OGM.

In TAM message playback mode: Press to go back to previous message.

MEMO/SKIP FORWARD ()

In Idle mode: Press and hold to record a memo.

In TAM message playback mode: Press to skip to next message.

DOWN/REDIAL LIST ()

In idle mode: Press to access the redial list.

In menu mode: Press to scroll down the menu items.

In Phonebook list / Redial list / Call list view mode: Press to scroll down the list.

In editing mode: Press to move the cursor one character to the right.

During a call or TAM message playback: Press to decrease the volume.

13-ANS ON/OFF ()

In Idle: Press to switch the answering machine ON or OFF.

14. PLAY/STOP ()

In idle mode: Press to playback messages.

During TAM message playback: Press to stop playing messages.

15. DELETE ()

In TAM message playback mode: Press to delete the current TAM message playback.

In idle mode: Press and hold to delete all old messages.

16. RECALL ()

During a call: Press to activate Recall for use with Telephone Network Services such as Call Waiting.

During message playback: Press to call back the caller if the caller's number is available

Handset

17. Incoming Call light

Flashes when receiving an incoming call.

18. Earpiece

19. Charger/Booster LED ()

- Steady red when the handset battery is charging.

- Long flashes in red when the handset battery is low.

- Flashes red quickly when handset registration is in progress.

- Flashes red slowly when the handset is not registered or not in charger.

- Steady blue when the booster function is activated.

20. Photo speed dial keys

This handset has 4 photo speed dial keys. These retrieve the same numbers stored in the Direct Memory keys of the AMPLIDECT COMBI 295.

Press to dial a pre-stored number.

21. Talk ON / Flash ()

- Make or answer a call.

- Green light shows when handset is in earpiece mode.

- During a call, Press to access Recall feature (R).

22. TALK OFF ()

- Hang up a call.

- Silence the ringer temporarily while the phone is ringing.

- Mute the PAGING TONE.

23. SPEAKERPHONE ()

- Make or answer a call using the speakerphone.

- Switch between the speakerphone and the earpiece.

- Light Orange when on speakerphone mode.

24. Microphone

25. Charging contacts

26. Speaker

27. Battery compartment cover

28. VOL+ ()


- Increase handset earpiece/ speaker phone volume during a call/ Increase Ringer volume in idle mode/ switch BOOSTER MODE on or off.

- Press and hold the key during a call to activate or no the receiver Booster function (LED bleu).


29. VOL- ()

Decrease handset earpiece / speaker phone volume during a call / Decrease / switch off RINGER

VOLUME in idle mode

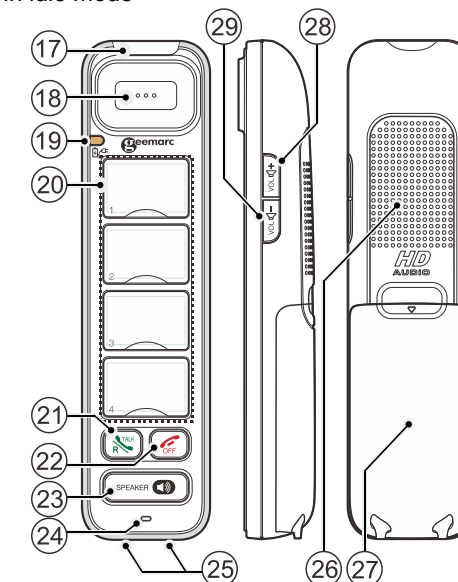

Insert Photos for speed dial keys

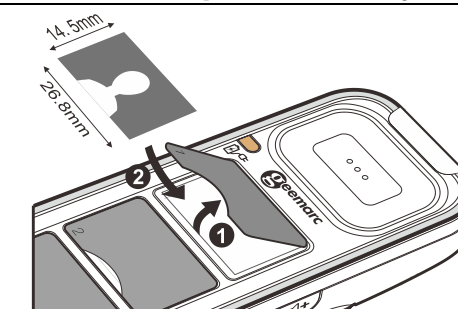



4

2

3

MENU SETTING




The phone settings are always set in the same manner:

Base: Press **Menu/OK**  button to open the main menu. Then use the **▼▲** buttons to select an entry and confirm it by pressing the **Menu/OK**  button.

The factory preset PIN code is **0000**

Example:

Set the language on the base

Press **MENU / OK**, , then use **▼▲** to select **LANGUAGE**, press **MENU / OK** , then use **▼▲** to select the desired language and then press **MENU / OK**  to confirm.

Base

PHONEBOOK

►VIEW►ADD►EDIT►DELETE►DELETE ALL►PB STATUS

CALL LOG: ►MISSED ►RECEIVED ►DIALLED

DATE & TIME: ►TIME FORMAT ►SET CLOCK

LANGUAGE

►ENGLISH ►DEUTSCH ►FRANÇAIS ►ITALIANO
►NEDERLANDS...

SETTINGS

►RING SETUP ►DELETE HS ►DIAL MODE ►FLASH TIME
►KEY TONE ►CHANGE PIN ►BS DEFAULT

PAGE ALL HS

REGISTRATION

ANS. MACHINE

►MSG PLAYBACK ►DELETE ALL ►MEMO ►TAM ON/OFF
►TAM SETTINGS (ANSWER MODE, OGM SETTINGS,
ANSWER DELAY, RECORD TIME, TAM LANGUAGE, BS
SCREENING, REMOTE ACC, COMPRESSION, CHANGE PIN,
TAM DEFAULT)

USING THE HANDSET

Booster mode

During a call, when in boost mode, you can increase the listening volume up to 30dB amplification.

- Press and Hold (2 seconds) the VOL+ button to activate Boost mode (LED will be steady blue).

- Press and hold again the VOL+ button to cancel Boost mode. (LED will go Off).

Storing a number to a picture memory button

On the base:

In idle mode: Dial your phone number and Press and hold the memory key until to hear a beep.

Using a picture memory

To make a call, remove the handset from the Pod and press any of these to dial a pre-stored number stored in the PHOTO and Memory keys of the AMPLIDECT COMBI PHOTO 295.

The [R/TALK] button (21) shows GREEN when the handset is in earpiece mode.

How to Receive an Incoming Call

If the handset is on the pod: Remove the handset from the pod, the call will automatically connect through the handset, and the [R/TALK] button (5) will show GREEN. If the handset not on the pod: Press the [R/TALK] button (21) to take the call. The [R/TALK] button (21) will show GREEN.

To finish the call, press the red [OFF] button, or place the handset back onto the pod.

How to Adjust the Handset Volume

During a call, you can increase or decrease the sound level of the handset earpiece by pressing the [VOL +] (28) and [VOL -] (29) buttons at the side of the handset.

[VOL +] (28) increases the volume.

[VOL -] (29) decreases the volume.


REGISTRATION & DE-REGISTRATION

Note: Handset and base station are pre-registered.

Up to four handsets can be registered to a single base station.

In the case of re-registration, the PIN code is **0000**.

Registration a photo handset

First place the base station of the AMPLIDECT COMBI 295 in Registration mode in select in press Menu REGISTRATION then press **MENU/OK** .

- PhotoDECT295 handset: Place the handset in the pod. The Charging/Booster/Registration LED (3) flashes RED to show registration in progress.

Registration can take up to 60 seconds. A confirmation tone is heard when registration is complete.




Repeat the procedure above if the handset remains unregistered.

Once charging and registration is complete, it will be possible to use the PhotoDECT295 to dial a pre-set number using one of its four photo speed dial keys, and to receive phone calls.

Note: if the handset doesn't arrive un registered, please follow this procedure:

Handset: Quickly press [VOL+] [SPK] [VOL-] [SPK] [VOL+] [VOL-] [SPK] in idle mode, then HS will have a "Bi,Bi" tone.

Registration a handset



On Handset: Press **MENU/OK** , select **▼▲** REGISTRATION, Press **MENU/OK** , select **▼▲**BASE(1-4), Press **MENU/OK**  enter PIN.


At the same time on **Base:** Press **MENU/OK** , select **▼▲** REGISTRATION, press **MENU/OK**  to see PLEASE WAIT

(If the registration is successful, there will be a confirmation beep).

De-Registration

On Base: Press **MENU/OK** , select **▼▲**SETTINGS, press **MENU/OK** .

select **▼▲**DELETE HS, press **MENU/OK** , enter PIN, press **MENU/OK** .

select **▼▲**one of the handsets that you want to de-register, press **MENU / OK**  to confirm.

INTERCOM

Internal Call




Base: Press **INT/☒/CLEAR** (*Int/☒/C*), use **▼▲** to select **HS 1 - 5**, press **MENU /OK**  to connect.

Handset: Function not available.

Call All Handsets

Base: Press **INT/☒/CLEAR** (*Int/☒/C*), use **▼▲** to select **ALL HANDSETS**, press **MENU /OK**  to connect.

Transfer an External Call

Base: Press **MENU /OK** , select **▼▲**handset number, press  or  on handset. After connection quit the call on base.

Conference Call


Base: Press **MENU / OK** .

Enter the desired handset number on keypad.

After connection press and hold  until CONFERENCE is shown.


ANSWERING MACHINE

The AMPLIDECT COMBI PHOTO 295 includes a telephone answering machine (TAM) to record messages (when it is on) on unanswered calls. It can store up to 59 messages (total recording time available is 15 minutes). As well as recording incoming messages, you can record memos for other users of your phone.

Press  **ANS ON/OFF** to switch the answering machine ON or OFF

Remote operations

You can access the TAM remotely from another tone-dialling phone to switch the TAM on and off and to access messages.

- Place a call from a tone-dialing phone.
- When the answering machine answers the call and starts playing OGM, press .
- Enter the 4-digit Remote Access PIN (factory default PIN is **0000**). Refer to the main user guide on how to set up your own remote access PIN.
- Press the following keys to carry out your desired function.

A	Functions	
	While message is not playing	While message is playing
2		Delete the current message playback
4		Repeat playing the current message from the beginning. Press twice to skip backward to the previous message.
5	Play the message	
6		Skip forward to play the next message
7	Turn on the answering machine	
8		Stop the current message playback
9	Turn off the answering machine	
*	Enter the 4-digit remote access PIN	

NOTE:

When the TAM memory is full (and answer mode is in ANS & RECORD), the answer mode will automatically change to ANSWER ONLY mode.

The TAM will automatically revert to ANS & RECORD mode, once some messages are deleted.

It is not possible to delete new (unplayed) messages.

MAINTENANCE

Clean the surface of the housing with a soft, fluff-free cloth. Do not use any cleaning agents or solvents.

GUARANTEE

From the moment your Geemarc AMPLIDECT™ COMBI-PHOTO 295 is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at **www.geemarc.com**.

The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

IMPORTANT : YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.


Please note: The guarantee applies to the United Kingdom only.

DECLARATIONS:

UKCA Directives

Geemarc Telecom SA hereby declares that this AMPLIDECT™ COMBI-PHOTO 295 is in compliance with the essential requirements and other relevant provisions of the Radio Equipment Directive 2014/53/UE.

The UKCA declaration of conformity may be consulted at **www.geemarc.com**

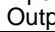RF power	< 250mW
Power Supply	
Base and Charger	Input 100-240V 50/60Hz; Output DC6V  0.45A, 2.7W adapter
Cordless Handset	2.4V, 500mAH rechargeable NiMH battery pack

Electrical connection:

The apparatus is designed to operate from a 230V 50Hz supply only. (Classified as hazardous voltage according to EN62368-1 standard). The apparatus does not incorporate an integral power on/off switch. To switch off the apparatus, disconnect the power adaptor.

When installing the apparatus, ensure that the mains power socket is readily accessible.

Telephone connection:

Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN62368-1 standard.

Recycling Directives

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip.

Deposit the product in an appropriate WEEE skip.

- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.



For product support and help visit our website at **www.geemarc.com**

E-mail : **help@geemarc.com**

telephone **01707 387602**

lines are open 09h00 to 16h00 Mon to Fri

Parc de l'Etoile, 2 Rue Galilée, 59760 Grande-Synthe, France.

Manufacturer: Vtech (Dongguan) Telecommunications Limited.
Vtech Science Park, Xia Ling Bei Management Zone, Liaobu, Dongguan, Guangdong.

This product is made in China To Geemarc Telecom



UGAMPLIDECT.COMBI-PHOTO.295_En_A3_V1.8